


TABLE TOP ETHYLENE OXIDE STERILIZERS MASA ÜSTÜ ETİLEN OKSİT STERİLİZATÖRÜ

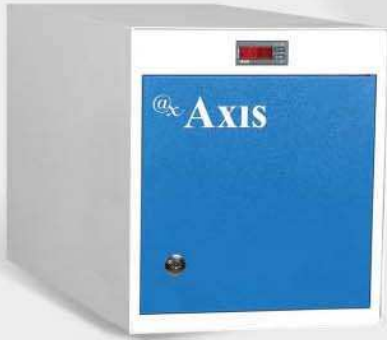

AX 60 / 60 Lt.



AX 135 / 135 Lt.



Door Lock System

**Sterilize with confidence by
Superior of Warm Sterilization at 54°C**

AAMI Standards

Sterilization Temperature	Sterilization Period (min)	Ventilation Period (min)
Room	12 hours	72 hours
37° C	4,5 hours	36 hours
54° C	1 hour	12 hours

Axis offers a new technology table top warm gas diffusion sterilization system are ideal for small office-based and medium sterilization requirements. Putting what ever you wish to be sterilized in to the Liner bag along with the Sterilizing Ampoule, Humidity Chip and close with string. When Liner Bag put in the sterilizer and snap the ampoule, the sterilization is begin. The sterilized materials are useable in 16 hours include sterilization and aeration period, with according to AAMI standards. It works with only 5 and 11 gr of ampoules. Simple to Install and Operate.

Kit bazlı gaz difüzyon teknolojisi ile yeni nesil gaz sterilizatörleridir. Sterilize edilecek malzemeler, pratik olarak sterilizasyon ampülü ve nem çipi ile birlikte Liner Bag içerisine konulur ve ağzı kapatılır. Liner bag, sterilizatörün içine yerleştirildikten sonra sterilizasyon ampülün başlığına basılarak sterilizasyon başlatılır. AAMI standartlarının doğrultusunda yaklaşık 16 saatlik sterilizasyon ve havalandırma sonrası malzemeleriniz kullanıma hazırdır. Oldukça düşük miktarlarda, 5 ve 11gr 'lık EO ampülleri ile çalışır. Montaj ve kullanım, pratik ve kolaydır.



Each kit includes Axis sterilizing gas ampoules, Liner Bag, Humidity chip and EO Spot sterilization indicator.

Her bir kit içerisinde, Sterilizasyon ampülü, Liner Bag, nem çipi ve EO sterilizasyon indikatörü bulunur.

MODEL	VOLUME KAPASİTE	IN / İÇ			OUT / DIŞ		
		Width En	Length Boy	Depth Derinlik	Width En	Length Boy	Depth Derinlik
AX 60	60 LT	35 cm	35 cm	49 cm	37 cm	41 cm	61 cm
AX 135	135 LT	35 cm	35 cm	110 cm	37 cm	41 cm	122 cm



EN ISO 13485



1015

Two years warranty. İki yıl garantili.

KORDON TIP SAĞLIK ARAÇLARI LTD. ŞTİ.

Tel. 0 (232) 348 60 60 (pbx)

Fax. 0 (232) 348 60 90

Web: www.kordontip.com - www.axis.com.tr