



Linear Bag 95-130 cm



100% EO Ampoule



Portable Sealed Container with Fully Automated Cartridge Activation



100% EO Cartridge



INDUSTRIAL ETHYLENE OXIDE STERILIZERS

Solutions



Sterilization Consumables



EO Monitor&Alarm



Call us for Conventional Type Industrial Sterilizers



AX 1000 / 1000 Lt.



KORDON TIP SAĞLIK ARAÇ GEREÇLERİ
MÜH. PROJE LTD. ŞTİ.

ADDRESS:
10006/1 Sokak No: 43 A.O.S.B. Çiğli - IZMIR
Tel. +90 (232) 348 60 60 (pbx)
Fax. +90 (232) 348 60 90
Web: <http://www.axis.com.tr>
<http://www.kordontip.com>
e-mail: info@kordontip.com

Distributor



- Fast Sterilization
- Practical, easy to use
- Robust design, maintenance free
- Safe
- Economic



AX 2000 / 2000 Lt.



The commands can be entered and viewed using the 7" touch screen color display.



All aspects of the sterilization (time, date, messages and warnings) are output to the mini printer.



AX 4000 / 4000 Lt.

- Fast Sterilization (4 hours)
- 100% EO
- 100% Safe
- Negative pressure throughout the cycle
- Real-time monitoring and pressure measurement
- Exhaust blockage warning
- Sterilization and Aeration in the same cabinet
- Microcomputer controlled
- Practical, easy to use
- Modular, ability to add new sterilization bags during operation
- Low gas consumption
- Robust construction, working without risk of failure
- Stainless steel chamber
- Decorative inox treated exterior
- Superior vacuum pump system
- Easy operation
- Easy installation; drainage, water or pressurised air are not required
- Easy to move, portable
- Bottom heating
- Simple to connect and change spare parts
- Leakproof door gasket and frame
- USB 2.0 connection



AX 6000 / 6000 Lt.

Axis offers the latest technology in industrial gas diffusion sterilization systems. This type of system is ideal for the requirements of industrial sterilisation. The items to be sterilised are placed into the liner bag along with the sterilizing ampoule, humidity chip and sealed with a tie-wrap. The sterilisation is started by simply pressing the ampoule and the sterilised materials are useable after the sterilisation and aeration period. A sterilisation cycle as little as 4 hours is available. Portable sealed containers with fully automated cartridge activation are available; avoiding the need for a liner-bag. Only 24 grams of ethylene oxide gas is used per cycle and there is no need for water, pressurised air or drainage lines, thus avoiding the need for heavy construction. Simple to install and operate – its robust construction means there are virtually no maintenance requirements and you can be assured of the reliability.

Operator safety is taken very seriously which is why the Axis system incorporates

- Fully negative pressure operating chamber
- Low gas consumption
- Gas diffusion liner bag system

MODEL	VOLUME	IN			OUT		
		Width	Length	Depth	Width	Length	Depth
AX 1000	1000 LT	83 cm	177 cm	70 cm	90 cm	207 cm	80 cm
AX 2000	2000 LT	159 cm	177 cm	70 cm	167 cm	207 cm	80 cm
AX 4000	4000 LT	165 cm	177 cm	136 cm	173 cm	207 cm	143 cm
AX 6000	6000 LT	248 cm	177 cm	136 cm	256 cm	207 cm	143 cm
AX 9000	9000 LT	374 cm	177 cm	136 cm	382 cm	207 cm	143 cm